



CoolSynth MIDI synthesizer Installation Guide

1st You are going so save the SF2 file in C:\brainm.20\Additional Downloads

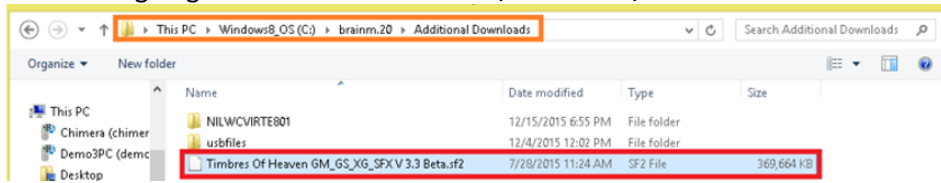


2nd The User Account Control Settings will open asking if you want the following program to make changes to your computer, click on 'Yes'


3rd Installer language: Please select a language then click 'Ok';

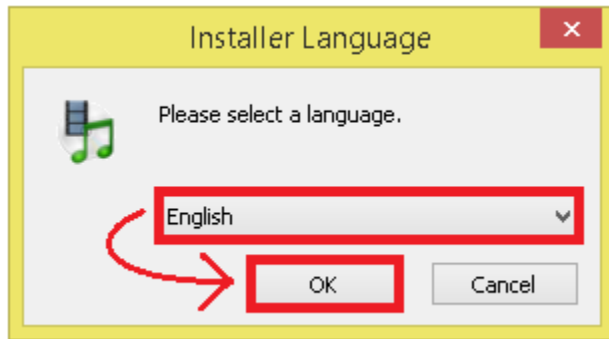

4th Welcome to the CoolSoft VirtualMIDISynth Setup. It is recommended that you close all other applications. After doing so click 'Next' to continue:

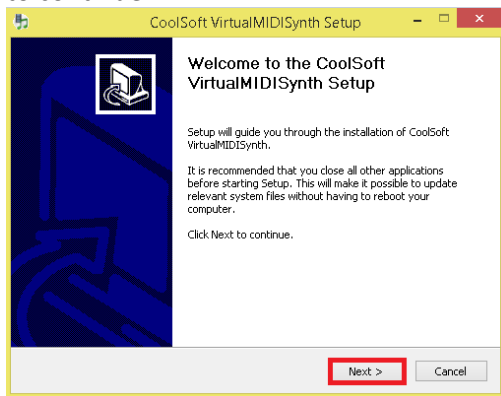


5th CoolSoft VirtualMIDISynth Setup will now look for a list of processes locking VirtualMIDISynth driver. If any processes are locking the driver you will have to right click to close them:

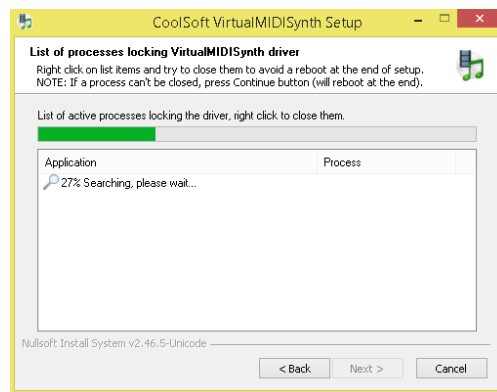



6th The release notes will then appear, to proceed click 'Install':

7th Once set has been completed successfully you will then click on 'Next':


CoolSynth MIDI synthesizer Installation Guide

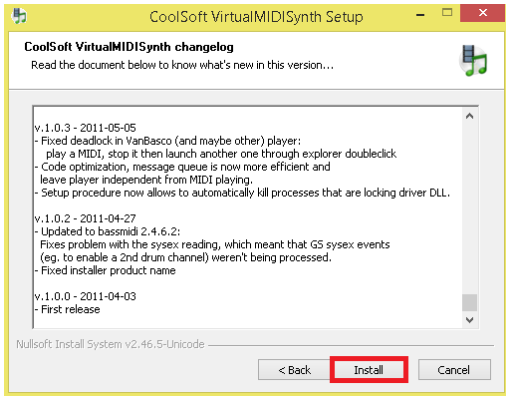


8th Un-check 'VirtualMIDISynth needs sound font files to work, open CoolSoft website to download them now', and 'Automatically check for updates' then click 'Next':

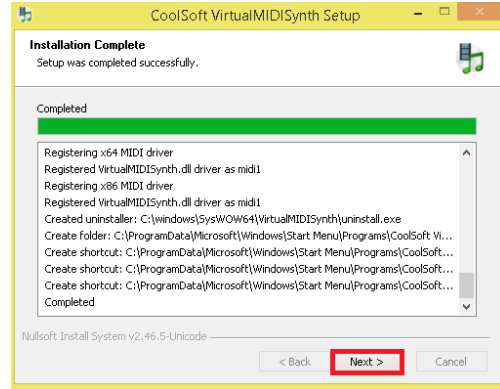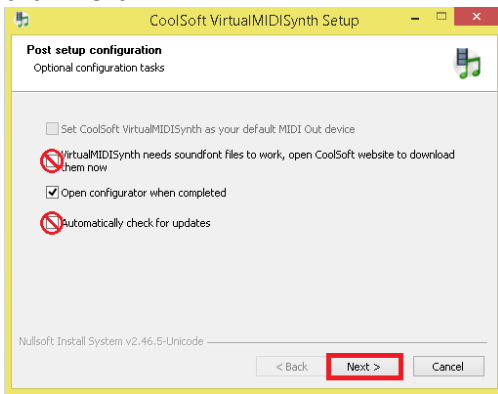


9th Click on the '+' button to search for the SF2 file that you saved in your Additional Downloads folder in step 1 of this instructional manual:

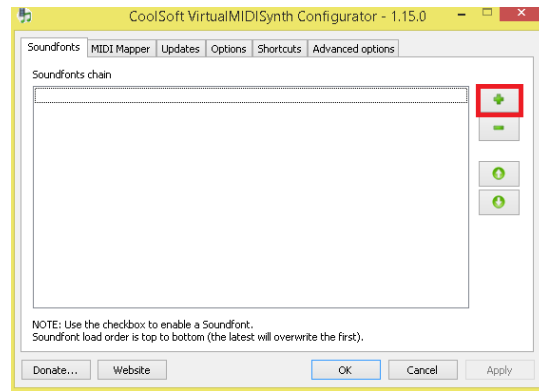

10th The SF2 file can be found in C:\brainm.20\Additional Downloads. Left Click on the file titled 'Timbres Of Heaven GM_GS_XG_SFX V 3.3 Beta' Then click on 'Open':

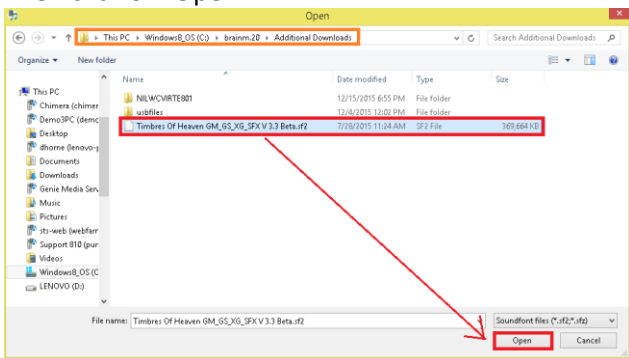

11th Click 'Apply' then 'OK':

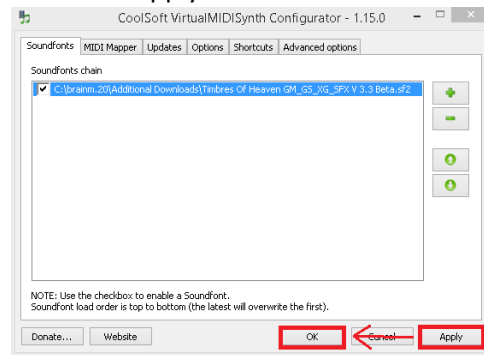

12th Click on 'Finish' to close Setup:

CoolSynth MIDI synthesizer Installation Guide

